

rejoin ϕ



EasyPass[®] Multi-Fire Suture Passer

High Efficiency

- ◇ Pass two sutures in articular with one-step suture loading

Simplicity

- ◇ Succinct and Ergonomic design allows for one-handed operation

Reliability

- ◇ Nitinol needle designed to maintain shape while flexing for precise control
- ◇ Jaw design grasps up to 12 mm of thick tissue

Low Profile

- ◇ Low profile design fits down a 5mm cannula



STEP 1 Examine EasyPass Multi-Fire Suture Passer and reset all the mechanisms. Then push down the mechanism ① to the position "UN-LOCK" to open the jaw, and push up the mechanism ② to the upper position. (Figure 1)


STEP 2 Load 1-2 inches of the #2 WilSuture Ultra into the slot on the bottom jaw. Grab both strands of WilSuture Ultra and pull them to the left-rear side (Figure 2). Push down on the thumbpad, the needle advances and the suture slides into the slot of the needle (Figure 3). Release the thumbpad. The needle will retract to its original position.

STEP 3 Load 1-2 inches of the other end of #2 WilSuture Ultra into the slot. Grab & pull both two strands of WilSuture Ultra to the right-rear side. (Figure 4)

STEP 4 Gently squeeze the handle until slight "Click" sound and see the 3 mm of the needle tip (Figure 5). Meanwhile the suture showed "left front right back" in slot (Figure 6). Release the handle and the needle will retract. Preloading of sutures is completed and ready for suture passing.

STEP 5 Pull the "trigger" to close the jaw, advance the Suture Passer through a lateral cannula and grasp on the tissue. Squeeze the handle until you can't squeeze it. Suture passing is completed. Release the handle and open the jaw, position the second stitch point, Squeeze the handle until you can't squeeze it. (Figure 7&8). The second suture passing is completed.

STEP 6 Retract the EasyPass Multi-Fire Suture Passer from the cannula, and push down the mechanism #1 to the position "UNLOCK". Pull all sutures into the upper jaw notch of the head, and remove all sutures.

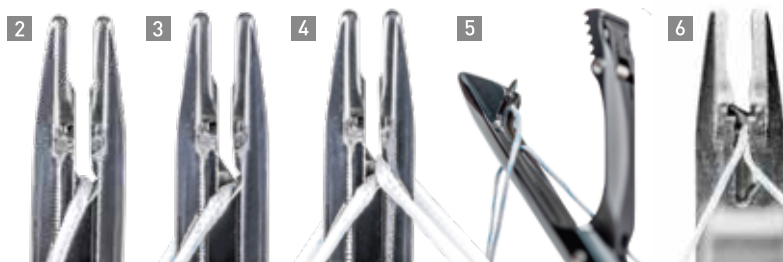

Hangzhou Rejoin Mastin Medical Device Co., Ltd

Headquarter Office: 22 Xinyan Rd, Yuhang Economic Development Zone, Hangzhou, Zhejiang, 311100, P.R. of China

Beijing Branch: Room 1605, Jinghui building, No. 118 Jianguo Road, Chaoyang District, Beijing, P.R. of China

Shanghai Branch: 309A, L1-B, Hongqiao World Center, 1588 Zhuguang Rd, Qingpu District, Shanghai, P.R. of China

contact@rejoin.com @rejoinmedical

Internal information is for professional reference only

rejoin 

www.rejoin-medical.com